



ACCESSIBLE PILL GRINDER INSTRUCTION MANUAL



Swallowing tablets isn't always easy. This stainless steel hand-crank pill grinder is designed to crush tablets into a fine or coarse powder so they're easier to mix with food, liquids, or tube feeds when appropriate.

With an adjustable grind setting, you can control the texture depending on what you need. The long crank handle makes grinding smoother and reduces hand strain compared to twist-style crushers, while the clear viewing window lets you see when you've reached the right consistency.

Solid, durable, and easy to clean, it's a practical tool for home use, caregiving, or travel.

Great for:

- Crushing tablets for easier swallowing
- Mixing medication into food or purees
- Preparing finely ground powder for tube feeding
- Caregivers managing daily medications
- Those with reduced grip strength
- Home or travel medication prep
- Travel or overnight stays
- Crushing pet tablets when prescribed

Features:

- Durable stainless steel body
- Hand-crank handle for smoother grinding
- Adjustable grind level from coarse to fine powder
- Clear window to monitor texture
- Large capacity chamber
- Compact and travel-friendly design
- Easy to disassemble and clean



ACCESSIBLE PILL GRINDER INSTRUCTION MANUAL

Image Description: A stainless steel cylindrical hand-crank pill grinder with a curved metal handle and black turning knob. A transparent window at the base shows crushed powder inside. Beside it is a small pile of finely ground powder to demonstrate consistency.

Details:

Material: Stainless Steel

Size: Please see options below...

Small: 12.5cm (L) x 4.5cm (diameter)

Large: 17.5cm (L) x 4.5cm (diameter)

Weight: 110 grams

Package Contains: 1x Pill Grinder

Age Guidance: Suitable for adult medication preparation. Use in line with prescribing guidance.

How to Use:

1. Make sure the grinder is clean and dry.
2. Place your tablet inside the chamber.
3. Adjust the grind setting. Tighter for a fine powder, looser for a coarser grind.
4. Screw the lid back on securely.
5. Turn the handle until the tablet is fully crushed.
6. Check the texture through the window.
7. Unscrew the base to collect the powder.

Care Instructions:

- Disassemble after use.
- Wash with warm soapy water.
- Dry completely before reassembling.

Important Notes:

- Not all medications can be crushed. Always check with your pharmacist or clinician first.
- Do not use with capsules, gel tablets, or modified-release medication unless approved.
- Clean thoroughly between different medications to avoid cross-contamination.
- Keep out of reach of children.