



HEATED SHOULDER BRACE (USB)

INSTRUCTION MANUAL



Tight, sore, or stiff shoulders can make everyday movement harder than it needs to be. This USB heated shoulder brace delivers targeted heat and gentle vibration to the shoulder joint, helping to ease tension, reduce discomfort, and support recovery in a practical, wearable way.

Designed for both short-term strain and ongoing joint conditions, it works across the muscles, tendons, and surrounding areas to bring warmth, relaxation, and a sense of support where you need it most.

Great for:

- Winding down sore or tired shoulders after a long day
- Relaxing in the evening when your body finally slows down
- Managing tension from stress, posture, or screen time
- Desk work or long hours sitting where shoulders start to tighten
- Car rides or travel where support and comfort make a difference
- Resting or taking pressure off during flare-ups
- Gentle support for ongoing joint conditions like arthritis or EDS
- Supporting recovery after activity, strain, or overuse
- Wearing while moving around at home without it slipping out of place
- Supporting stiffness, limited movement, or general discomfort
- People who benefit from heat and vibration for muscle relaxation
- Wrapping around other areas like elbows, knees, or arms when needed



HEATED SHOULDER BRACE (USB)

INSTRUCTION MANUAL

Features:

- 3 heat levels so you can choose what feels right for your body
- 3 vibration modes for gentle muscle relaxation
- Simple control panel with temperature and battery display
- Adjustable cross-body strap design for a secure, personalised fit
- Low-profile design that can be worn under or over clothing
- USB rechargeable with no need for disposable batteries
- 5000mAh battery offering up to approx 75 minutes of use
- Approx 1 hour recharge time
- Maximum temperature up to approx 75°C
- Flexible wrap design that can be used on multiple body areas

Image Description: A series of images showing a person wearing the heated shoulder brace from front, side, and back views, demonstrating how it sits over one shoulder and secures across the chest and back with adjustable straps. Close-up images highlight the heating panel, breathable material, stitching, and control panel. Additional images show the brace laid flat to display its full shape, straps, and included components.

Details:

Material: Neoprene

Size: 45cm x 32cm x 1cm

Battery Capacity: 5000mAh

Modes: 3 heat levels, 3 vibration modes

Maximum Temperature: Approx 75°C

Weight: 350g

Package Contains: 1x Shoulder Wrap, USB Charging Cord, User Manual

Age Guidance: Suitable for teens and adults. Use with supervision if supporting younger users.

How to use:

1. Connect USB to a power source or power bank
2. Wrap over your shoulder and across your body, adjusting straps to suit
3. Turn on and choose your preferred heat and or vibration setting
4. Use for 20 to 30 minutes at a time to support skin safety
5. Turn off after use and allow to cool before removing or storing



HEATED SHOULDER BRACE (USB)

INSTRUCTION MANUAL

Care Instructions:

- Wipe clean with a damp cloth only
- Do not immerse in water
- Allow to fully dry before next use
- Store in a cool, dry place
- Check regularly for signs of wear or damage

Safety Information:

- Do not use while sleeping
- Always turn off and unplug after use
- Monitor skin during use to protect skin integrity
- Not recommended for reduced skin sensation without medical advice
- Discontinue use if damaged or faulty
- Keep out of reach of children unless supervised